



## ABO and Rh DISCREPANCY WORKSHEET

MH.TS.TE.221A.06

Effective Date: 10/8/2015

Page 1 of 1

### Patient Information

|                 |                  |                     |             |            |
|-----------------|------------------|---------------------|-------------|------------|
| <b>Name</b>     | <b>MRN</b>       | <b>FIN</b>          | <b>Acc#</b> | <b>DOB</b> |
| Collection Date | Previous /ABO/Rh | Previous Antibodies |             | Diagnosis  |

| Initial Testing    | Forward Group  |        |          |        | Reverse Group |          |         |        | Rh                   |         |                 |      | Antibody Screen |      |         |        |        |
|--------------------|----------------|--------|----------|--------|---------------|----------|---------|--------|----------------------|---------|-----------------|------|-----------------|------|---------|--------|--------|
| Phase              | AntiA          | Anti-B | Anti-A,B | Interp | A1 Cells      | A2 Cells | B Cells | Interp | Anti-D               | Rh cont | Weak D          | CC   | Interp          | SC I | SC II   | Auto   | Interp |
| ProVue/Vision      |                |        |          |        |               |          |         |        |                      |         |                 |      |                 |      |         |        |        |
| Immediate Spin     |                |        |          |        |               |          |         |        |                      |         |                 |      |                 |      |         |        |        |
| RT Incubation      |                |        |          |        |               |          |         |        | Include Auto Control |         |                 |      |                 |      |         |        |        |
| 4□ C Incubation    |                |        |          |        |               |          |         |        | Include Auto Control |         |                 |      |                 |      |         |        |        |
| Additional Testing | Anti-A1 Lectin |        | DAT      |        |               |          |         |        |                      |         | Mini Cold Panel |      |                 |      |         |        |        |
| Phase              | Result         | Interp | Poly     | CC     | Interp        | IgG      | CC      | Interp | C3                   | C3 CC   | Interp          | SC I | SC II           | Auto | I-/cord | Interp |        |
| Immediate Spin     |                |        |          |        |               |          |         |        |                      |         |                 |      |                 |      |         |        |        |
| RT Incubation      |                |        |          |        |               |          |         |        |                      |         |                 |      |                 |      |         |        |        |
| 4□ C Incubation    |                |        |          |        |               |          |         |        |                      |         |                 |      |                 |      |         |        |        |

Use of common data elements is not endorsed by the American Medical Association

**Action(s)**

- Antibody screen **(and auto control)** performed at room temperature to evaluate potential interference by auto/allo antibody(ies).
- Rouleaux detected. Patient testing performed using saline replacement technique.
- Patient is Group A or AB and A subgroup suspected. Patient tested with anti-A<sub>1</sub> lectin and A<sub>2</sub> cells.
- Cold antibody /auto antibody suspected. Mini cold panel performed.
- DAT performed
- Forward/reverse type discrepancy due to age/immuno-suppression. Reverse type **(include auto control)** performed at RT and/or 4° C.
- Forward/reverse type discrepancy due to stem cell transplant. Reverse type **(include auto control)** performed at RT and/or at 4° C.

**Conclusion/Comments**

| Performed |      | Reviewed |      |
|-----------|------|----------|------|
| By        | Date | By       | Date |
|           |      |          |      |

Use of commendable practice does not ensure conformance with AABB Standards